








Ampoule Adapter

Safe and quick aspiration, assures optimal quantity aspiration without the use of needles, followed with an immediate mixing with a drug vial.

Instructions

		
<p>1. Break ampoule and insert into Ampulmix</p>	<p>2. Connect syringe to Ampulmix</p>	<p>3. Aspirate diluent into syringe</p>
		
<p>4. Pull out ampoule holder</p>	<p>5. Pierce drug vial and inject the diluent</p>	<p>6. Swirl and aspirate ready drug into syringe</p>
		
<p>7. Disconnect vial adapter and assemble needle</p>		